

DIRECT REPLY ON WS1:

1. Connect **AUXILIARY GTB – RED SIDE** (+24Vdc) to **GTB007R** (7th screw from bottom on right side of GTB)
2. Connect **GTB008R** (8th screw from bottom on right side of GTB) to **GTB034R** (34th screw from bottom on right side of GTB)
3. Connect **GTB031R** (31st screw from bottom on right side of GTB) to **AUXILIARY GTB – BLACK SIDE** (0Vdc).

DIRECT REPLY ON WS2:

1. Connect **AUXILIARY GTB – RED SIDE** (+24Vdc) to **GTB013R** (13th screw from bottom on right side of GTB)
2. Connect **GTB014R** (14th screw from bottom on right side of GTB) to **GTB034R** (34th screw from bottom on right side of GTB)
3. Connect **GTB031R** (31st screw from bottom on right side of GTB) to **AUXILIARY GTB – BLACK SIDE** (0Vdc).

DIRECT REPLY ON WS3:

1. Connect **AUXILIARY GTB – RED SIDE** (+24Vdc) to **GTB019R** (19th screw from bottom on right side of GTB)
2. Connect **GTB020R** (20th screw from bottom on right side of GTB) to **GTB034R** (34th screw from bottom on right side of GTB)
3. Connect **GTB031R** (31st screw from bottom on right side of GTB) to **AUXILIARY GTB – BLACK SIDE** (0Vdc).

