
 odel  x-ray	Odel X-ray	EEN0015 Eng. to Eng.Note 2006-02
	GILARDONI DOSEMAT Chamber connection	

R306. 4x – “PROGRAM H. F. US” generator series

Odel provides a direct connection with Gilardoni DOSEMAT chambers.

R306. 3x – “ENDEAVOUR” generator series

Connections are explained in Technical Manual AEC pre-amplifier schematic section as shown below:

MCU connector	DOSEMAT connector	CHAMBER symbol	MCU symbol	WIRE COLOR
8	---	---	---	---
7	3P1	III -	DX-	YELLOW
6	2P1	II -	SX-	GREY
5	1P1	I -	CX-	BROWN
4	6P1	I +	CX+	WHITE
3	4P1	III +	DX+	GREEN
2	5P1	II +	SX+	PINK
1	---	---	---	---

Connections must be on:

MCU.J12 connector for Detector number 1 option

MCU.J13 connector for Detector number 2 option

DOCUMENTO		FOGLIO
NOMEFILE		1
<small> \ISO9000\25 EEN-015 Gilardoni DOSEMAT chamber connection.doc </small>		di 1