


When you place an order to Odel for a printer you are sure that Cable is tested, Cable is right positioned inside the control console and correctly connected to EXPU.
The printer is the correct one: Trademark, Model and Interface type.
The printer and cable is tested directly connected to your control console and your generator.

Cable is ODEL cable type 429494 dwg 7118

Tutti i diritti sono espressamente riservati. Il presente documento non può essere copiato, fotocopiato, riprodotto o altro modo, né in parte né integralmente, senza previa autorizzazione della IDEL S.p.A.

All rights strictly reserved. No part of this document can be copied, photocopied, reproduced or used in any way without prior permission from IDEL S.p.A.

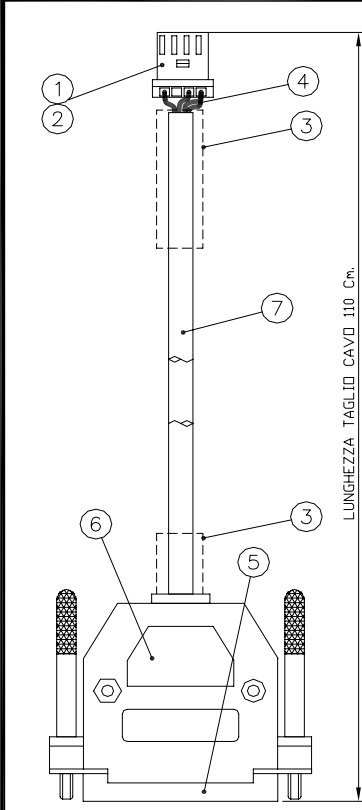


LUNGHEZZA TAGLIO CAVO 110 Cm.

NOTE PER LA SALDATURA DEI CONNETTORI:

SPELATURA CAVO 3cm AMBO LE PARTI


(3) Sul cavo: termorestringente da 6mm lunghezza= 3 Cm.

VISTO DA SOPRA - DIETRO (LATO DEL CAVO)

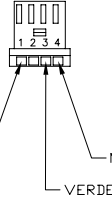

GIALLO — (2) GIALLO
MARRONE/BIANCO — (7) MARRONE/BIANCO
VERDE — (3) VERDE

VISTO DAL RETRO (LATO DEL CAVO)

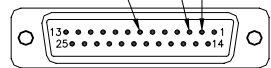


(3) Sul cavo: termorestringente da 6mm lunghezza= 3 Cm.
(4) Sul filo: termorestringente da 2mm lunghezza= 1 Cm.

1	CAVO SCHERMATO INTRECCIATO	7	110 Cm.	-	-	307.310
1	GUSCIO PER CONN. DB25 M	6	-	-	-	132.820
1	CONNETTORE DB25 M sald.	5	-	-	-	132.818
6	TERMORESTRINGENTE ø2	4	1 Cm.	-	-	176.413
2	TERMORESTRINGENTE ø6	3	3 Cm.	-	-	178.416
3	CONTATTI PER CONNETTORE AMP	2	-	-	-	172.306
1	CONNETTORE AMP 4 POLI	1	-	-	-	132.303
N	DESCRIZIONE	POS.	MISURA	FORO	TRATTAMENTO	CODICE

DATA	FIRMA	REVISIONI	2	3	4	5	6	7	8	9
DISEGNATO	31/01/2002	N. MOD.	-							
CONTROLLATO	31/01/2002	DATA	-							
APPROVATO	31/01/2002	FIRMA								

Il presente disegno e' di proprieta' della ditta "Odel S.p.A." o tutti gli effetti di legge



odel x-ray

429494.7118.08

CAVO DA PCU
A STAMPANTE PER ENDEAVOUR

SOSTITUISCE

DATA

FIRMA

MATERIALE

TRATTAMENTO

SCALA

TOLLERANZA

FOGLIO di FOGLI

VERSIONE

1

INDICE di REVISIONE

①	2	3	4	5	6	7	8	9
---	---	---	---	---	---	---	---	---

1:1

1

1 di 1

DOCUMENTO	FOGLIO
	1
	di
	2

Dedicated to all those who "I can't stand doing it by myself"




odel x-ray

Odel X-ray

SERIAL PRINTER CONNECTION on VirUS CONTROL CONSOLE

EEN-020
Eng. to Eng. Note
2007-06

Connected on EXPU on J14

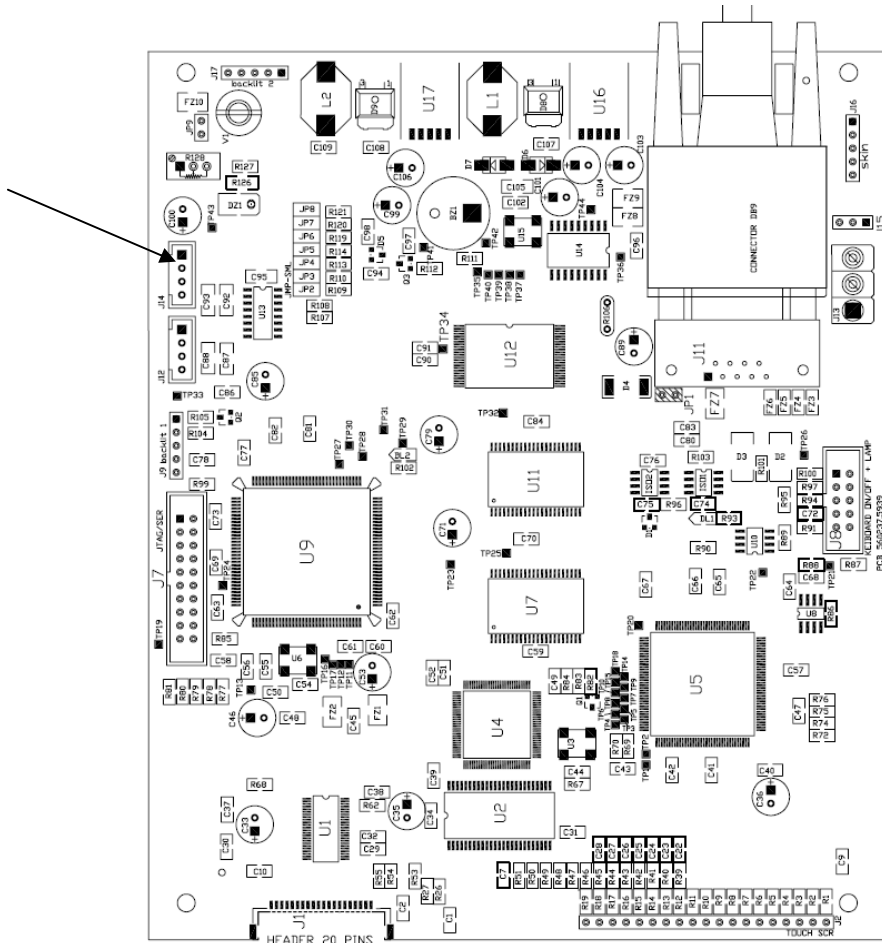

Printer is :
 Trademark: EPSON
 Actual Model: ET-TM-U220B-007
 Interface type: Serial Communication Printer

NOTE:
 Printing methods are written on USER MANUAL

DOCUMENTO		FOGLIO
		2
Dedicated to all those who "I can't stand doing it by myself"		di 2