



odel x-ray

Odel X-ray


HSS Modification Note for VARIAN STATORS

EEN-026
Eng. to Eng. Note
2009-04


Necessary Item:

n.1 220pF ceramic capacitor 36Vdc minimum voltage

Varian Statots requires an 220pF filter capacitor on short circuit current sensing circuit according to following illustrations:

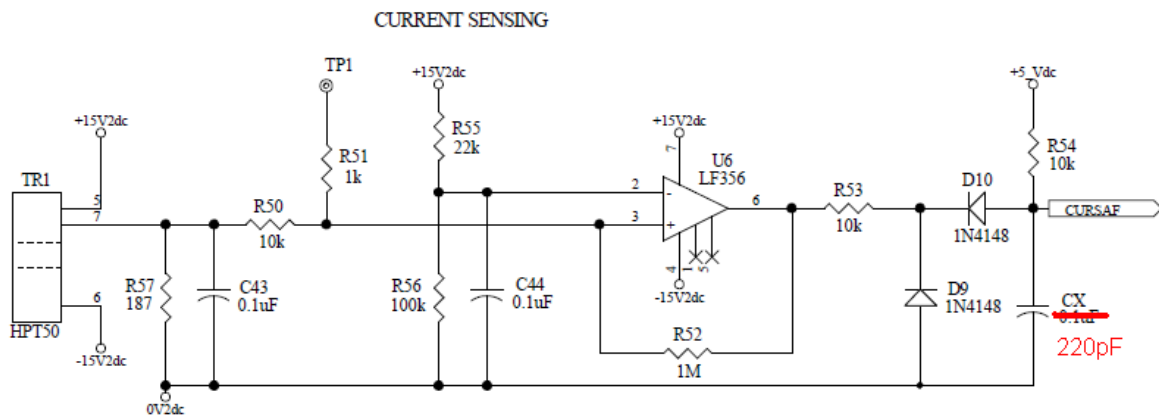

Locate the current sensing section on HSS control board and solder the capacitor directly on the diodes anodes as per following picture:

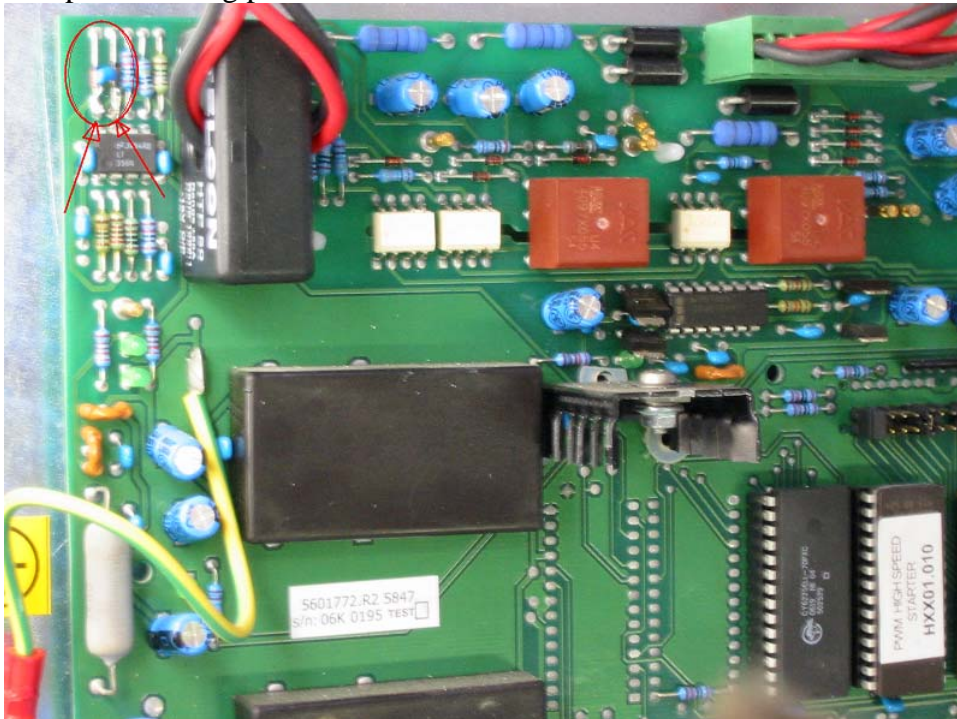

| DOCUMENTO | | FOGLIO |
|---|--|--------|
| NOMEFILE | | 1 |
| Errore. Nome della proprietà del documento sconosciuto. EEN-026 V1 R1 HSS MODIFICATION FOR VARIAN TUBE.doc | | di |
| | | 1 |